

source



SCIIN

UG003 SCIIN Force

User Instructions

Order yours today:

T. 01276 676 006

E. info@sourcesupplies.co.uk

W. sourcesupplies.co.uk

reduce + reuse + recycle

User instructions

How to wear

Hold the snood and pull down over your head so it sits around your neck. Then pull the snood up so it covers your nose and mouth. If you wish to cover your ears too simply pull up over your ears and tuck over your ears if required. When not in use simply slip the snood down around your neck or pull the snood up off your head. (see diagram). Ensure your nose, mouth and ears are covered for maximum protection.

Care guidelines

We recommend hand-washing your mask for up to 15 washes.

Do not tumble dry as this will cause the material to shrink and damage the antiviral coating. Allow to air dry naturally on a flat surface.

To remove creases you can iron the product on a low settings (silk) making sure the logo is face down and no steam is used.

Keep your snood in a cool dry place when not in use and/or in the original packaging.

Best practise

Avoid touching your face after removing your mask and always wash your hands to avoid transferring any germs.

Disposal




The material is 100% biodegradable and safe to be disposed of in a normal waste bin. The packaging is 100% recyclable. Please dispose of responsibly to protect the environment.

Antiviral coating

The antiviral coating provides 99% effective protection from new. After 15 hand-washes the coating is still 80%+ effective according to Polygiene's tests. Polygiene's antimicrobial technology prevents growth of microorganisms in the fibres (such as bacteria, fungi and parasites) maintaining a cleaner fabric for longer, and the non-necessity for frequent washing.

Allergies

ViralOff is safe and has no detrimental effect on your skin nor does it interfere with the skin's natural bacterial flora.

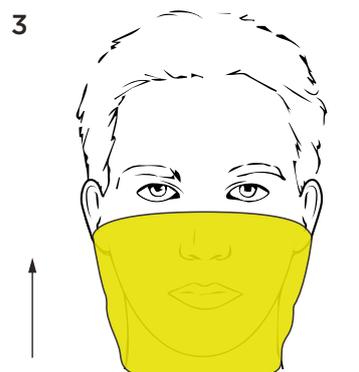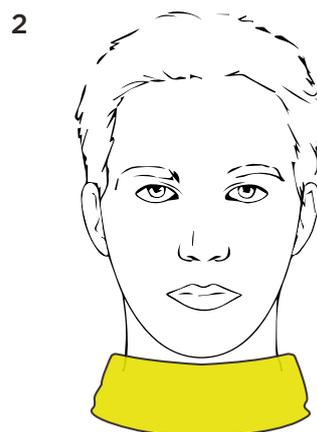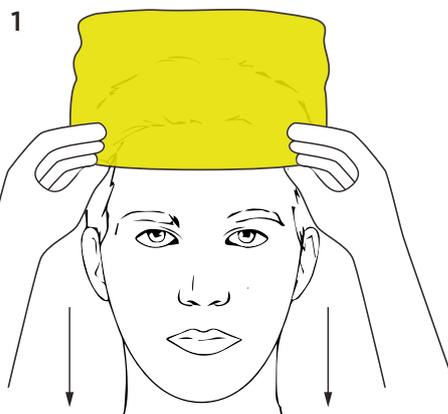

Frequently Asked Questions

Can I wear the Sciin Force both ways?

Yes. The Sciin Force has coating on both sides so it doesn't reduce the antiviral effect whichever way you wear it.

Does the Sciin Force lose its shape?

The Sciin Force is made from lightweight bamboo viscose for increased comfort and breathability so you will see a slight stretch over time depending on how long you wear the mask. Hand washing and allowing to dry naturally on a flat surface will help return the mask back to its original shape and size.

Can a child wear a Sciin Force?

Sciin Force is available in child sizes on request, please email us at hello@sciin.co.uk to enquire.

Can I wash my Sciin Force?

We recommend a maximum of 15 hand washes in warm water without detergent.

How often should I wash my Sciin Force?

You can wear your Sciin Force for as long as you like before having to wash it and you do not have to wash the Sciin Force after each wear, however we do recommend washing if the fabric becomes dirty. The antiviral coating provides 99% effective protection from new. After 15 hand-washes the coating is still 80%+ effective.

Can I dry my Sciin Force on a radiator or in a tumble dryer?

We do not advise drying on a radiator or in a tumble dryer and recommend leaving to air dry naturally on a flat surface. High temperatures can damage the antiviral coating and reduce its effectiveness.

Can I iron my Sciin Force?

We do not recommend ironing Sciin Force but if you wish to remove any creases we recommend ironing with the logo face down on the lowest setting (silk) and no steam is used. High temperatures can damage its antiviral coating.

Can I dry clean my Sciin Force?

No. We do not recommend dry cleaning your Sciin Force.

Will it provide 100% protection against viruses?

No device can guarantee 100% protection against viruses. The antiviral coating provides 99% effective protection from new. After 15 hand-washes the coating is still 80%+ effective.

How else can I protect myself against viruses?

Wash your hands frequently with soap and clean water. Cough and sneeze into a tissue and dispose of it immediately. Avoid touching your eyes, ears, nose and mouth.